

Table of contents

\\USER

<table border="1"><tr><td>MRS</td></tr></table>	MRS
MRS	

<table border="1"><tr><td><table border="1"><tr><td>Rizor_UCSB_Final</td></tr></table></td></tr></table>	<table border="1"><tr><td>Rizor_UCSB_Final</td></tr></table>	Rizor_UCSB_Final
<table border="1"><tr><td>Rizor_UCSB_Final</td></tr></table>	Rizor_UCSB_Final	
Rizor_UCSB_Final		

<table border="1"><tr><td><table border="1"><tr><td><table border="1"><tr><td>UCSB_Rizor_Final</td></tr></table></td></tr></table></td></tr></table>	<table border="1"><tr><td><table border="1"><tr><td>UCSB_Rizor_Final</td></tr></table></td></tr></table>	<table border="1"><tr><td>UCSB_Rizor_Final</td></tr></table>	UCSB_Rizor_Final
<table border="1"><tr><td><table border="1"><tr><td>UCSB_Rizor_Final</td></tr></table></td></tr></table>	<table border="1"><tr><td>UCSB_Rizor_Final</td></tr></table>	UCSB_Rizor_Final	
<table border="1"><tr><td>UCSB_Rizor_Final</td></tr></table>	UCSB_Rizor_Final		
UCSB_Rizor_Final			

<table border="1"><tr><td><table border="1"><tr><td><table border="1"><tr><td>localizer</td></tr></table></td></tr></table></td></tr></table>	<table border="1"><tr><td><table border="1"><tr><td>localizer</td></tr></table></td></tr></table>	<table border="1"><tr><td>localizer</td></tr></table>	localizer
<table border="1"><tr><td><table border="1"><tr><td>localizer</td></tr></table></td></tr></table>	<table border="1"><tr><td>localizer</td></tr></table>	localizer	
<table border="1"><tr><td>localizer</td></tr></table>	localizer		
localizer			

<table border="1"><tr><td><table border="1"><tr><td><table border="1"><tr><td>t1_mp2rage_sag_p3_0.75mm</td></tr></table></td></tr></table></td></tr></table>	<table border="1"><tr><td><table border="1"><tr><td>t1_mp2rage_sag_p3_0.75mm</td></tr></table></td></tr></table>	<table border="1"><tr><td>t1_mp2rage_sag_p3_0.75mm</td></tr></table>	t1_mp2rage_sag_p3_0.75mm
<table border="1"><tr><td><table border="1"><tr><td>t1_mp2rage_sag_p3_0.75mm</td></tr></table></td></tr></table>	<table border="1"><tr><td>t1_mp2rage_sag_p3_0.75mm</td></tr></table>	t1_mp2rage_sag_p3_0.75mm	
<table border="1"><tr><td>t1_mp2rage_sag_p3_0.75mm</td></tr></table>	t1_mp2rage_sag_p3_0.75mm		
t1_mp2rage_sag_p3_0.75mm			

<table border="1"><tr><td><table border="1"><tr><td><table border="1"><tr><td>ep2d_bold_1.5mm_MB4_HCP_AP_CMRR_3TE_1.5mm</td></tr></table></td></tr></table></td></tr></table>	<table border="1"><tr><td><table border="1"><tr><td>ep2d_bold_1.5mm_MB4_HCP_AP_CMRR_3TE_1.5mm</td></tr></table></td></tr></table>	<table border="1"><tr><td>ep2d_bold_1.5mm_MB4_HCP_AP_CMRR_3TE_1.5mm</td></tr></table>	ep2d_bold_1.5mm_MB4_HCP_AP_CMRR_3TE_1.5mm
<table border="1"><tr><td><table border="1"><tr><td>ep2d_bold_1.5mm_MB4_HCP_AP_CMRR_3TE_1.5mm</td></tr></table></td></tr></table>	<table border="1"><tr><td>ep2d_bold_1.5mm_MB4_HCP_AP_CMRR_3TE_1.5mm</td></tr></table>	ep2d_bold_1.5mm_MB4_HCP_AP_CMRR_3TE_1.5mm	
<table border="1"><tr><td>ep2d_bold_1.5mm_MB4_HCP_AP_CMRR_3TE_1.5mm</td></tr></table>	ep2d_bold_1.5mm_MB4_HCP_AP_CMRR_3TE_1.5mm		
ep2d_bold_1.5mm_MB4_HCP_AP_CMRR_3TE_1.5mm			

<table border="1"><tr><td><table border="1"><tr><td><table border="1"><tr><td>TR3000_TE60SpinEchoFieldMap_AP_CMRR</td></tr></table></td></tr></table></td></tr></table>	<table border="1"><tr><td><table border="1"><tr><td>TR3000_TE60SpinEchoFieldMap_AP_CMRR</td></tr></table></td></tr></table>	<table border="1"><tr><td>TR3000_TE60SpinEchoFieldMap_AP_CMRR</td></tr></table>	TR3000_TE60SpinEchoFieldMap_AP_CMRR
<table border="1"><tr><td><table border="1"><tr><td>TR3000_TE60SpinEchoFieldMap_AP_CMRR</td></tr></table></td></tr></table>	<table border="1"><tr><td>TR3000_TE60SpinEchoFieldMap_AP_CMRR</td></tr></table>	TR3000_TE60SpinEchoFieldMap_AP_CMRR	
<table border="1"><tr><td>TR3000_TE60SpinEchoFieldMap_AP_CMRR</td></tr></table>	TR3000_TE60SpinEchoFieldMap_AP_CMRR		
TR3000_TE60SpinEchoFieldMap_AP_CMRR			

<table border="1"><tr><td><table border="1"><tr><td><table border="1"><tr><td>TR3000_TE60SpinEchoFieldMap_PA_CMRR</td></tr></table></td></tr></table></td></tr></table>	<table border="1"><tr><td><table border="1"><tr><td>TR3000_TE60SpinEchoFieldMap_PA_CMRR</td></tr></table></td></tr></table>	<table border="1"><tr><td>TR3000_TE60SpinEchoFieldMap_PA_CMRR</td></tr></table>	TR3000_TE60SpinEchoFieldMap_PA_CMRR
<table border="1"><tr><td><table border="1"><tr><td>TR3000_TE60SpinEchoFieldMap_PA_CMRR</td></tr></table></td></tr></table>	<table border="1"><tr><td>TR3000_TE60SpinEchoFieldMap_PA_CMRR</td></tr></table>	TR3000_TE60SpinEchoFieldMap_PA_CMRR	
<table border="1"><tr><td>TR3000_TE60SpinEchoFieldMap_PA_CMRR</td></tr></table>	TR3000_TE60SpinEchoFieldMap_PA_CMRR		
TR3000_TE60SpinEchoFieldMap_PA_CMRR			

\\USER\MRS\Rizor_UCSB_Final\UCSB_Rizor_Final\localizer

TA: 0:15 PM: REF Voxel size: 0.5×0.5×5.0 mmPAT: Off Rel. SNR: 1.00 : qfl

Properties

Prio recon	On
Load images to viewer	On
Inline movie	Off
Auto store images	On
Load images to stamp segments	On
Load images to graphic segments	On
Auto open inline display	Off
Auto close inline display	Off
Start measurement without further preparation	Off
Wait for user to start	Off
Start measurements	Single measurement

Routine

Slice group	1
Slices	1
Dist. factor	20 %
Position	Isocenter
Orientation	Sagittal
Phase enc. dir.	A >> P
Slice group	2
Slices	1
Dist. factor	20 %
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	A >> P
Slice group	3
Slices	1
Dist. factor	20 %
Position	Isocenter
Orientation	Coronal
Phase enc. dir.	R >> L
AutoAlign	---
Phase oversampling	0 %
FoV read	250 mm
FoV phase	100.0 %
Slice thickness	5.0 mm
TR	8.6 ms
TE	3.69 ms
Averages	2
Concatenations	3
Filter	Elliptical filter
Coil elements	AC

Contrast - Common

TR	8.6 ms
TE	3.69 ms
TD	0 ms
MTC	Off
Magn. preparation	None
Flip angle	20 deg
Fat suppr.	None
Water suppr.	None
SWI	Off

Contrast - Dynamic

Averages	2
Averaging mode	Short term
Reconstruction	Magnitude
Measurements	1

Contrast - Dynamic

Multiple series	Each measurement
-----------------	------------------

Resolution - Common

FoV read	250 mm
FoV phase	100.0 %
Slice thickness	5.0 mm
Base resolution	256
Phase resolution	100 %
Phase partial Fourier	Off
Interpolation	On

Resolution - iPAT

PAT mode	None
----------	------

Resolution - Filter Image

Image Filter	Off
Distortion Corr.	Off
Prescan Normalize	Off
Normalize	Off
B1 filter	Off

Resolution - Filter Rawdata

Raw filter	Off
Elliptical filter	On

Geometry - Common

Slice group	1
Slices	1
Dist. factor	20 %
Position	Isocenter
Orientation	Sagittal
Phase enc. dir.	A >> P
Slice group	2
Slices	1
Dist. factor	20 %
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	A >> P
Slice group	3
Slices	1
Dist. factor	20 %
Position	Isocenter
Orientation	Coronal
Phase enc. dir.	R >> L
FoV read	250 mm
FoV phase	100.0 %
Slice thickness	5.0 mm
TR	8.6 ms
Multi-slice mode	Sequential
Series	Interleaved
Concatenations	3

Geometry - AutoAlign

Slice group	1
Position	Isocenter
Orientation	Sagittal
Phase enc. dir.	A >> P
Slice group	2
Position	Isocenter
Orientation	Transversal

Geometry - AutoAlign

Phase enc. dir.	A >> P
Slice group	3
Position	Isocenter
Orientation	Coronal
Phase enc. dir.	R >> L
AutoAlign	---
Initial Position	Isocenter
L	0.0 mm
P	0.0 mm
H	0.0 mm
Initial Rotation	0.00 deg
Initial Orientation	Coronal

Geometry - Saturation

Saturation mode	Standard
Fat suppr.	None
Water suppr.	None
Special sat.	None

Geometry - Tim Planning Suite

Set-n-Go Protocol	Off
Table position	H
Table position	0 mm
Inline Composing	Off

Geometry - Tim CT

Tim CT mode	Off
Slices	1
Slice thickness	5.0 mm
Dist. factor	20 %
FoV read	250 mm
FoV phase	100.0 %
Segments	1

System - Miscellaneous

Positioning mode	REF
Table position	F
Table position	0 mm
MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	F >> H
Coil Combine Mode	Adaptive Combine
Save uncombined	Off
Matrix Optimization	Off
AutoAlign	---
Coil Select Mode	Off - AutoCoilSelect

System - Adjustments

B0 Shim mode	Tune up
B1 Shim mode	TrueForm
Confirm freq. adjustment	Off
Assume Dominant Fat	Off
Assume Silicone	Off
Adjustment Tolerance	Auto

System - Adjust Volume

Position	Isocenter
Orientation	Transversal
Rotation	0.00 deg
A >> P	263 mm
R >> L	350 mm
F >> H	350 mm
Reset	Off

System - pTx Volumes

B1 Shim mode	TrueForm
Excitation	Slice-sel.

System - Tx/Rx

Frequency 1H	296.904869 MHz
Correction factor	1
Gain	High
Img. Scale Cor.	1.000
Reset	Off
? Ref. amplitude 1H	0.000 V

Physio - Signal1

1st Signal/Mode	None
TR	8.6 ms
Concatenations	3
Segments	1

Physio - Cardiac

Tagging	None
Magn. preparation	None
Fat suppr.	None
Dark blood	Off
FoV read	250 mm
FoV phase	100.0 %
Phase resolution	100 %

Physio - PACE

Resp. control	Off
Concatenations	3

Inline - Common

Subtract	Off
Measurements	1
StdDev	Off
Liver registration	Off
Save original images	On

Inline - MIP

MIP-Sag	Off
MIP-Cor	Off
MIP-Tra	Off
MIP-Time	Off
Save original images	On

Inline - Soft Tissue

Wash - In	Off
Wash - Out	Off
TTP	Off
PEI	Off
MIP - time	Off
Measurements	1

Inline - Composing

Inline Composing	Off
Distortion Corr.	Off

Inline - MapIt

Save original images	On
MapIt	None
Flip angle	20 deg
Measurements	1
Contrasts	1
TR	8.6 ms

Inline - MapIt

TE	3.69 ms
----	---------

Sequence - Part 1

Introduction	On
Dimension	2D
Phase stabilisation	Off
Asymmetric echo	Allowed
Contrasts	1
Flow comp.	No
Multi-slice mode	Sequential
Bandwidth	320 Hz/Px

Sequence - Part 2

Segments	1
Acoustic noise reduction	Active
RF pulse type	Normal
Gradient mode	Normal
Excitation	Slice-sel.
RF spoiling	On

Sequence - Assistant

Mode	Off
------	-----

\\USER\MRS\Rizor_UCSB_Final\UCSB_Rizor_Final\t1_mp2rage_sag_p3_0.75mm

TA: 8:50 PM: REF Voxel size: 0.8×0.8×0.8 mmPAT: 3 Rel. SNR: 1.00 : tfl

Properties

Prio recon	Off
Load images to viewer	On
Inline movie	Off
Auto store images	On
Load images to stamp segments	Off
Load images to graphic segments	Off
Auto open inline display	Off
Auto close inline display	Off
Start measurement without further preparation	Off
Wait for user to start	Off
Start measurements	Single measurement

Routine

Slab group	1
Slabs	1
Dist. factor	50 %
Position	Isocenter
Orientation	Sagittal
Phase enc. dir.	A >> P
AutoAlign	Head > Basis
Phase oversampling	0 %
Slice oversampling	8.3 %
Slices per slab	192
FoV read	240 mm
FoV phase	93.8 %
Slice thickness	0.75 mm
TR	4300.0 ms
TE	2.27 ms
Averages	1
Concatenations	1
Filter	None
Coil elements	AC

Contrast - Common

TR	4300.0 ms
TE	2.27 ms
Magn. preparation	Non-sel. IR
TI 1	1000 ms
TI 2	3200 ms
Flip angle 1	4.0 deg
Flip angle 2	4.0 deg
Fat suppr.	Water excit. fast
Water suppr.	None

Contrast - Dynamic

Averages	1
Averaging mode	Long term
Reconstruction	Magnitude
Measurements	1
Multiple series	Each measurement

Resolution - Common

FoV read	240 mm
FoV phase	93.8 %
Slice thickness	0.75 mm
Base resolution	320
Phase resolution	100 %
Slice resolution	100 %
Phase partial Fourier	Off

Resolution - Common

Slice partial Fourier	6/8
Interpolation	Off

Resolution - iPAT

PAT mode	GRAPPA
Accel. factor PE	3
Ref. lines PE	36
Accel. factor 3D	1
Reference scan mode	Integrated

Resolution - Filter Image

Image Filter	Off
Distortion Corr.	Off
Prescan Normalize	Off
Normalize	Off
B1 filter	Off

Resolution - Filter Rawdata

Raw filter	Off
Elliptical filter	Off

Geometry - Common

Slab group	1
Slabs	1
Dist. factor	50 %
Position	Isocenter
Orientation	Sagittal
Phase enc. dir.	A >> P
Slice oversampling	8.3 %
Slices per slab	192
FoV read	240 mm
FoV phase	93.8 %
Slice thickness	0.75 mm
TR	4300.0 ms
Multi-slice mode	Single shot
Series	Interleaved
Concatenations	1

Geometry - AutoAlign

Slab group	1
Position	Isocenter
Orientation	Sagittal
Phase enc. dir.	A >> P
AutoAlign	Head > Basis
Initial Position	Isocenter
L	0.0 mm
P	0.0 mm
H	0.0 mm
Initial Rotation	0.00 deg
Initial Orientation	Sagittal

Geometry - Navigator**Geometry - Tim Planning Suite**

Set-n-Go Protocol	Off
Table position	H
Table position	0 mm
Inline Composing	Off

System - Miscellaneous

Positioning mode	REF
Table position	H
Table position	0 mm
MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	F >> H
Coil Combine Mode	Sum of Squares
Save uncombined	Off
Matrix Optimization	Off
AutoAlign	Head > Basis
Coil Select Mode	Default

System - Adjustments

B0 Shim mode	Brain
B1 Shim mode	TrueForm
Confirm freq. adjustment	Off
Assume Dominant Fat	Off
Assume Silicone	Off
Adjustment Tolerance	Auto

System - Adjust Volume

Position	Isocenter
Orientation	Sagittal
Rotation	0.00 deg
A >> P	225 mm
F >> H	240 mm
R >> L	144 mm
Reset	Off

System - pTx Volumes

B1 Shim mode	TrueForm
Excitation	Non-sel.

System - Tx/Rx

Frequency 1H	296.904869 MHz
Correction factor	1
Gain	High
Img. Scale Cor.	1.000
Reset	Off
? Ref. amplitude 1H	0.000 V

Physio - Signal1

1st Signal/Mode	None
TR	4300.0 ms
Concatenations	1

Physio - Cardiac

Magn. preparation	Non-sel. IR
TI 1	1000 ms
TI 2	3200 ms
Fat suppr.	Water excit. fast
Dark blood	Off
FoV read	240 mm
FoV phase	93.8 %
Phase resolution	100 %

Physio - PACE

Resp. control	Off
Concatenations	1

Inline - Common

Subtract	Off
----------	-----

Inline - Common

Measurements	1
StdDev	Off
Save original images	On

Inline - MIP

MIP-Sag	Off
MIP-Cor	Off
MIP-Tra	Off
MIP-Time	Off
Save original images	On

Inline - Composing

Inline Composing	Off
Distortion Corr.	Off

Inline - MapIt

Save original images	On
MapIt	None
Flip angle 1	4.0 deg
Flip angle 2	4.0 deg
Measurements	1
TR	4300.0 ms
TE	2.27 ms

Sequence - Part 1

Introduction	On
Dimension	3D
Elliptical scanning	Off
Reordering	Linear
Asymmetric echo	Allowed
Flow comp.	No
Multi-slice mode	Single shot
Echo spacing	7.2 ms
Bandwidth	200 Hz/Px

Sequence - Part 2

RF pulse type	Fast
Gradient mode	Fast*
Excitation	Non-sel.
RF spoiling	On
Incr. Gradient spoiling	Off
Turbo factor	156

Sequence - Assistant

Mode	Off
------	-----

\\USER\MRS\Rizor_UCSB_Final\UCSB_Rizor_Final\lep2d_bold_1.5mm_MB4_HCP_AP_CMRR_3TE_1.5mm
TA: 10:54 PM: REF Voxel size: 2.0×2.0×1.5 mmPAT: 3 Rel. SNR: 1.00 : epfid

Properties

Prio recon	Off
Load images to viewer	On
Inline movie	Off
Auto store images	On
Load images to stamp segments	Off
Load images to graphic segments	Off
Auto open inline display	Off
Auto close inline display	Off
Start measurement without further preparation	Off
Wait for user to start	On
Start measurements	Single measurement

Routine

Slice group	1
Slices	100
Dist. factor	0 %
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	P >> A
AutoAlign	---
Phase oversampling	0 %
FoV read	200 mm
FoV phase	102.0 %
Slice thickness	1.50 mm
TR	2100 ms
TE 1	15.20 ms
TE 2	33.87 ms
TE 3	52.54 ms
Multi-band accel. factor	4
Filter	Raw filter
Coil elements	AC

Contrast - Common

TR	2100 ms
TE 1	15.20 ms
TE 2	33.87 ms
TE 3	52.54 ms
MTC	Off
Magn. preparation	None
Flip angle	70 deg
Fat suppr.	Fat sat.

Contrast - Dynamic

Averaging mode	Long term
Reconstruction	Magn./Phase
Measurements	300
Delay in TR	0 ms
Multiple series	Off

Resolution - Common

FoV read	200 mm
FoV phase	102.0 %
Slice thickness	1.50 mm
Base resolution	100
Phase resolution	100 %
Phase partial Fourier	Off
Interpolation	Off

Resolution - iPAT

PAT mode	GRAPPA
Accel. factor PE	3
Ref. lines PE	42
Reference scan mode	FLEET

Resolution - Filter Image

Distortion Corr.	Off
Prescan Normalize	Off

Resolution - Filter Rawdata

Raw filter	On
Elliptical filter	Off
Hamming	Off

Geometry - Common

Slice group	1
Slices	100
Dist. factor	0 %
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	P >> A
FoV read	200 mm
FoV phase	102.0 %
Slice thickness	1.50 mm
TR	2100 ms
Multi-slice mode	Interleaved
Series	Interleaved
Multi-band accel. factor	4

Geometry - AutoAlign

Slice group	1
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	P >> A
AutoAlign	---
Initial Position	Isocenter
L	0.0 mm
P	0.0 mm
H	0.0 mm
Initial Rotation	-180.00 deg
Initial Orientation	Transversal

Geometry - Saturation

Fat suppr.	Fat sat.
Special sat.	None

Geometry - Tim Planning Suite

Set-n-Go Protocol	Off
Table position	H
Table position	0 mm
Inline Composing	Off

System - Miscellaneous

Positioning mode	REF
Table position	H
Table position	0 mm
MSMA	S - C - T
Sagittal	R >> L

System - Miscellaneous

Coronal	A >> P
Transversal	F >> H
Coil Combine Mode	Sum of Squares
Matrix Optimization	Off
AutoAlign	---
Coil Select Mode	Default

System - Adjustments

B0 Shim mode	Brain
B1 Shim mode	TrueForm
Confirm freq. adjustment	Off
Assume Dominant Fat	Off
Assume Silicone	Off
Adjustment Tolerance	Auto

System - Adjust Volume

Position	Isocenter
Orientation	Transversal
Rotation	270.00 deg
R >> L	200 mm
A >> P	204 mm
F >> H	150 mm
Reset	Off

System - pTx Volumes

B1 Shim mode	TrueForm
Excitation	Standard

System - Tx/Rx

Frequency 1H	296.904869 MHz
Correction factor	1
Gain	High
Img. Scale Cor.	1.000
Reset	Off
? Ref. amplitude 1H	0.000 V

Physio - Signal1

1st Signal/Mode	None
TR	2100 ms
Multi-band accel. factor	4

BOLD

GLM Statistics	Off
Dynamic t-maps	Off
Ignore meas. at start	0
Ignore after transition	0
Model transition states	On
Temp. highpass filter	On
Threshold	4.00
Paradigm size	20
Meas[1]	Baseline
Meas[2]	Baseline
Meas[3]	Baseline
Meas[4]	Baseline
Meas[5]	Baseline
Meas[6]	Baseline
Meas[7]	Baseline
Meas[8]	Baseline
Meas[9]	Baseline
Meas[10]	Baseline
Meas[11]	Active
Meas[12]	Active
Meas[13]	Active
Meas[14]	Active

BOLD

Meas[15]	Active
Meas[16]	Active
Meas[17]	Active
Meas[18]	Active
Meas[19]	Active
Meas[20]	Active
Motion correction	Off
Spatial filter	Off
Measurements	300
Delay in TR	0 ms
Multiple series	Off

Sequence - Part 1

Introduction	Off
Contrasts	3
Flow comp.	No
Multi-slice mode	Interleaved
Free echo spacing	Off
Echo spacing	0.53 ms
Bandwidth	2380 Hz/Px

Sequence - Part 2

EPI factor	102
Gradient mode	Fast
Excitation	Standard
RF spoiling	Off

Sequence - Special

Excite pulse duration	5720 us
FLEET ref. prep. scans	0
FLEET ref. min. TR	0 ms
Inter-TE delay	0 us
Single-band images	On
MB LeakBlock kernel	On
MB dual kernel	Off
MB RF phase scramble	On
SENSE1 coil combine	Off
Invert RO/PE polarity	Off
Force Maxwell corr.	Off
Disable freq. update	Off
Force equal slice timing	Off
Online multi-band recon.	Online
FFT scale factor	1.00
Fat saturation FA	110.0 deg
FLEET iPAT ref. FA	12.0 deg
Physio recording	DICOM
Triggering scheme	Standard

\\USER\MRS\Rizor_UCSB_Final\UCSB_Rizor_Final\TR3000_TE60SpinEchoFieldMap_AP_CMRR

TA: 1:15 PM: FIX Voxel size: 2.0×2.0×1.5 mmPAT: 2 Rel. SNR: 1.00 : epse

Properties

Prio recon	Off
Load images to viewer	On
Inline movie	Off
Auto store images	On
Load images to stamp segments	Off
Load images to graphic segments	Off
Auto open inline display	Off
Auto close inline display	Off
Start measurement without further preparation	Off
Wait for user to start	Off
Start measurements	Single measurement

Resolution - iPAT

Ref. lines PE	24
Reference scan mode	Single-shot

Resolution - Filter Image

Distortion Corr.	Off
Prescan Normalize	Off

Resolution - Filter Rawdata

Raw filter	On
Elliptical filter	Off
Hamming	Off

Routine

Slice group	1
Slices	100
Dist. factor	0 %
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	A >> P
AutoAlign	Head > Brain
Phase oversampling	0 %
FoV read	192 mm
FoV phase	100.0 %
Slice thickness	1.50 mm
TR	3000 ms
TE	60.00 ms
Multi-band accel. factor	5
Filter	Raw filter
Coil elements	AC

Geometry - Common

Slice group	1
Slices	100
Dist. factor	0 %
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	A >> P
FoV read	192 mm
FoV phase	100.0 %
Slice thickness	1.50 mm
TR	3000 ms
Multi-slice mode	Interleaved
Series	Interleaved
Multi-band accel. factor	5

Geometry - AutoAlign

Slice group	1
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	A >> P
AutoAlign	Head > Brain
Initial Position	Isocenter
L	0.0 mm
P	0.0 mm
H	0.0 mm
Initial Rotation	0.00 deg
Initial Orientation	Transversal

Contrast - Common

TR	3000 ms
TE	60.00 ms
MTC	Off
Magn. preparation	None
Flip angle	90 deg
Refocus flip angle	180 deg
Fat suppr.	None
Grad. rev. fat suppr.	Disabled

Contrast - Dynamic

Averaging mode	Long term
Reconstruction	Magnitude
Measurements	3
Delay in TR	0 ms
Multiple series	Off

Resolution - Common

FoV read	192 mm
FoV phase	100.0 %
Slice thickness	1.50 mm
Base resolution	96
Phase resolution	100 %
Phase partial Fourier	7/8
Interpolation	Off

Resolution - iPAT

PAT mode	GRAPPA
Accel. factor PE	2

Geometry - Saturation

Fat suppr.	None
Grad. rev. fat suppr.	Disabled
Special sat.	None

Geometry - Tim Planning Suite

Set-n-Go Protocol	Off
Table position	H
Table position	0 mm
Inline Composing	Off

System - Miscellaneous

Positioning mode	FIX
Table position	H
Table position	0 mm
MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	F >> H
Coil Combine Mode	Sum of Squares

System - Miscellaneous

Matrix Optimization	Off
AutoAlign	Head > Brain
Coil Select Mode	Default

System - Adjustments

B0 Shim mode	Advanced
B1 Shim mode	TrueForm
Confirm freq. adjustment	Off
Assume Dominant Fat	Off
Assume Silicone	Off
Adjustment Tolerance	Auto

System - Adjust Volume

Position	Isocenter
Orientation	Transversal
Rotation	0.00 deg
A >> P	192 mm
R >> L	192 mm
F >> H	150 mm
Reset	Off

System - pTx Volumes

B1 Shim mode	TrueForm
Excitation	Standard

System - Tx/Rx

Frequency 1H	296.904869 MHz
Correction factor	1
Gain	High
Img. Scale Cor.	1.000
Reset	Off
? Ref. amplitude 1H	0.000 V

Physio - Signal1

1st Signal/Mode	None
TR	3000 ms
Multi-band accel. factor	5

BOLD

GLM Statistics	Off
Dynamic t-maps	Off
Ignore meas. at start	0
Ignore after transition	0
Model transition states	On
Temp. highpass filter	On
Threshold	4.00
Paradigm size	20
Meas[1]	Baseline
Meas[2]	Baseline
Meas[3]	Baseline
Meas[4]	Baseline
Meas[5]	Baseline
Meas[6]	Baseline
Meas[7]	Baseline
Meas[8]	Baseline
Meas[9]	Baseline
Meas[10]	Baseline
Meas[11]	Active
Meas[12]	Active
Meas[13]	Active
Meas[14]	Active
Meas[15]	Active
Meas[16]	Active
Meas[17]	Active

BOLD

Meas[18]	Active
Meas[19]	Active
Meas[20]	Active
Motion correction	Off
Spatial filter	Off
Measurements	3
Delay in TR	0 ms
Multiple series	Off

Sequence - Part 1

Introduction	Off
Contrasts	1
Multi-slice mode	Interleaved
Free echo spacing	Off
Echo spacing	0.62 ms
Bandwidth	1930 Hz/Px

Sequence - Part 2

EPI factor	96
Gradient mode	Normal
Excitation	Standard

Sequence - Special

Excite pulse duration	5120 us
Refocus pulse duration	10240 us
Single-band images	On
MB LeakBlock kernel	On
MB dual kernel	Off
MB RF phase scramble	Off
Time-shifted MB RF	Off
SENSE1 coil combine	Off
Invert RO/PE polarity	Off
PF omits higher k-space	Off
Disable freq. update	Off
Force equal slice timing	Off
Online multi-band recon.	Online
FFT scale factor	1.00
Physio recording	Off
Triggering scheme	Standard

\\USER\MRS\Rizor_UCSB_Final\UCSB_Rizor_Final\TR3000_TE60SpinEchoFieldMap_PA_CMRR

TA: 1:15 PM: FIX Voxel size: 2.0×2.0×1.5 mmPAT: 2 Rel. SNR: 1.00 : epse

Properties

Prio recon	Off
Load images to viewer	On
Inline movie	Off
Auto store images	On
Load images to stamp segments	Off
Load images to graphic segments	Off
Auto open inline display	Off
Auto close inline display	Off
Start measurement without further preparation	Off
Wait for user to start	Off
Start measurements	Single measurement

Resolution - iPAT

Ref. lines PE	24
Reference scan mode	Single-shot

Resolution - Filter Image

Distortion Corr.	Off
Prescan Normalize	Off

Resolution - Filter Rawdata

Raw filter	On
Elliptical filter	Off
Hamming	Off

Routine

Slice group	1
Slices	100
Dist. factor	0 %
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	A >> P
AutoAlign	Head > Brain
Phase oversampling	0 %
FoV read	192 mm
FoV phase	100.0 %
Slice thickness	1.50 mm
TR	3000 ms
TE	60.00 ms
Multi-band accel. factor	5
Filter	Raw filter
Coil elements	AC

Geometry - Common

Slice group	1
Slices	100
Dist. factor	0 %
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	A >> P
FoV read	192 mm
FoV phase	100.0 %
Slice thickness	1.50 mm
TR	3000 ms
Multi-slice mode	Interleaved
Series	Interleaved
Multi-band accel. factor	5

Geometry - AutoAlign

Slice group	1
Position	Isocenter
Orientation	Transversal
Phase enc. dir.	A >> P
AutoAlign	Head > Brain
Initial Position	Isocenter
L	0.0 mm
P	0.0 mm
H	0.0 mm
Initial Rotation	0.00 deg
Initial Orientation	Transversal

Contrast - Common

TR	3000 ms
TE	60.00 ms
MTC	Off
Magn. preparation	None
Flip angle	90 deg
Refocus flip angle	180 deg
Fat suppr.	None
Grad. rev. fat suppr.	Disabled

Contrast - Dynamic

Averaging mode	Long term
Reconstruction	Magnitude
Measurements	3
Delay in TR	0 ms
Multiple series	Off

Resolution - Common

FoV read	192 mm
FoV phase	100.0 %
Slice thickness	1.50 mm
Base resolution	96
Phase resolution	100 %
Phase partial Fourier	7/8
Interpolation	Off

Resolution - iPAT

PAT mode	GRAPPA
Accel. factor PE	2

Geometry - Saturation

Fat suppr.	None
Grad. rev. fat suppr.	Disabled
Special sat.	None

Geometry - Tim Planning Suite

Set-n-Go Protocol	Off
Table position	H
Table position	0 mm
Inline Composing	Off

System - Miscellaneous

Positioning mode	FIX
Table position	H
Table position	0 mm
MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	F >> H
Coil Combine Mode	Sum of Squares

System - Miscellaneous

Matrix Optimization	Off
AutoAlign	Head > Brain
Coil Select Mode	Default

System - Adjustments

B0 Shim mode	Advanced
B1 Shim mode	TrueForm
Confirm freq. adjustment	Off
Assume Dominant Fat	Off
Assume Silicene	Off
Adjustment Tolerance	Auto

System - Adjust Volume

Position	Isocenter
Orientation	Transversal
Rotation	0.00 deg
A >> P	192 mm
R >> L	192 mm
F >> H	150 mm
Reset	Off

System - pTx Volumes

B1 Shim mode	TrueForm
Excitation	Standard

System - Tx/Rx

Frequency 1H	296.904869 MHz
Correction factor	1
Gain	High
Img. Scale Cor.	1.000
Reset	Off
? Ref. amplitude 1H	0.000 V

Physio - Signal1

1st Signal/Mode	None
TR	3000 ms
Multi-band accel. factor	5

BOLD

GLM Statistics	Off
Dynamic t-maps	Off
Ignore meas. at start	0
Ignore after transition	0
Model transition states	On
Temp. highpass filter	On
Threshold	4.00
Paradigm size	20
Meas[1]	Baseline
Meas[2]	Baseline
Meas[3]	Baseline
Meas[4]	Baseline
Meas[5]	Baseline
Meas[6]	Baseline
Meas[7]	Baseline
Meas[8]	Baseline
Meas[9]	Baseline
Meas[10]	Baseline
Meas[11]	Active
Meas[12]	Active
Meas[13]	Active
Meas[14]	Active
Meas[15]	Active
Meas[16]	Active
Meas[17]	Active

BOLD

Meas[18]	Active
Meas[19]	Active
Meas[20]	Active
Motion correction	Off
Spatial filter	Off
Measurements	3
Delay in TR	0 ms
Multiple series	Off

Sequence - Part 1

Introduction	Off
Contrasts	1
Multi-slice mode	Interleaved
Free echo spacing	Off
Echo spacing	0.62 ms
Bandwidth	1930 Hz/Px

Sequence - Part 2

EPI factor	96
Gradient mode	Normal
Excitation	Standard

Sequence - Special

Excite pulse duration	5120 us
Refocus pulse duration	10240 us
Single-band images	On
MB LeakBlock kernel	On
MB dual kernel	Off
MB RF phase scramble	Off
Time-shifted MB RF	Off
SENSE1 coil combine	Off
Invert RO/PE polarity	On
PF omits higher k-space	Off
Disable freq. update	Off
Force equal slice timing	Off
Online multi-band recon.	Online
FFT scale factor	1.00
Physio recording	Off
Triggering scheme	Standard